

Instruction for use :
 Consult Instruction for Use

APPLICATION - MINDRAY BS-120 / BS-200 / BS-200E - PROPOSAL

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use.

BS120 / BS200 / BS200E

<u>PARAMETERS</u>			
Test No.	BIDI **	R1	200 / 200 / 200
Full Name	Bili D	R2	50 / 50 / 50
Standard No.	1	Sample volume	25 / 25 / 25
Reac. Type	Endpoint	R1 Blank	
Pri. Wavelength	546 nm / 546 nm / 546 nm	Mixed Rgt Blank	
Sec. Wavelength		Linearity Range	*
Direction	Increase	Linearity Limit	*
Reac Time	0 20	Substrate Limit	*
Incuba. Time	3	Factor	
Units	mg/dL	Prozone check	*
Precision	0.01	q1	Q2
		q3	Q4
		PC	Abs
<u>CALIBRATION (Cal + Rgt Blk)</u>			
Rule	Linear one point / Linear two point / Linear two point		
Sensitivity	1		
Replicates	2		
Interval (days)	0		
Difference Limit			
SD			
Blank Response			
Error Limit			
Correlation coefficient	1		

*Enter by the user.