

Instruction for use :
 Consult Instruction for Use

APPLICATION - MINDRAY BS-120 / BS-200 / BS-200E - PROPOSAL

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use.

BS120 / BS200 / BS200E

<u>PARAMETERS</u>			
Test No.	ICRP **	R1	300
Full Name	CRP	R2	30
Standard No.	6	Sample volume	19
Reac. Type	Endpoint	R1 Blank	
Pri. Wavelength	340 nm	Mixed Rgt Blank	
Sec. Wavelength		Linearity Range	*
Direction	Increase	Linearity Limit	*
Reac Time	-1 20	Substrate Limit	*
Incuba. Time	10	Factor	
Units	mg/L	Prozone check	*
Precision	0.1	q1	Q2
		q3	Q4
		PC	Abs
<u>CALIBRATION (Cal + Rgt Blk)</u>			
Rule	Logit-Log5		
Sensitivity	1		
Replicates	2		
Interval (days)	0		
Difference Limit			
SD			
Blank Response			
Error Limit			
Correlation coefficient	1		

*Enter by the user.