

Instruction for use :  
 Consult Instruction for Use

**APPLICATION - MINDRAY BS-120 / BS-200 / BS-200E - PROPOSAL**

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use.

Calibrator, Controls and sample must be  
 prediluted 1/10

**BS120 / BS200 / BS200E**

<u>PARAMETERS</u>			
Test No.	TRF2**	R1	190
Full Name	TRF	R2	38
Standard No.	6	Sample volume	3
Reac. Type	Endpoint	R1 Blank	
Pri. Wavelength	340 nm	Mixed Rgt Blank	
Sec. Wavelength		Linearity Range	*
Direction	Increase	Linearity Limit	*
Reac Time	-1 20	Substrate Limit	*
Incuba. Time	10	Factor	
Units	mg/dL	Prozone check	*
Precision	0	q1	Q2
		q3	Q4
		PC	Abs
<u>CALIBRATION (Cal + Rgt Blk)</u>			
Rule	Logit-Log5		
Sensitivity	1		
Replicates	2		
Interval (days)	0		
Difference Limit			
SD			
Blank Response			
Error Limit			
Correlation coefficient	1		

\*Enter by the user.