

Consult Instruction for Use

APPLICATION - Mindray - BS 360 - PROPOSAL

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use.

Define / Edite Chemestries

Chem	CHDL	N°		Sample Type	Serum
Chemistry	HDL	Print name			
Reaction Type	Endpoint	Reaction Direction	Positive		
Pri Wave	570 nm	Sec Wave			
Unit	mg/dL	Decimal	0		
Incubation Time	28				
Blank time	-3		-1		
Reaction Time	16			18	
Sample Vol	Aspirated	Diluent	Reagent Vol		
Standard	2 <input type="text"/> μ L	<input type="text"/> μ L	<input type="text"/> μ L	R1	150 <input type="text"/> μ L
Decreased	<input type="text"/> μ L	<input type="text"/> μ L	<input type="text"/> μ L	R2	50 <input type="text"/> μ L
Increased	<input type="text"/> μ L	<input type="text"/> μ L	<input type="text"/> μ L		
<input type="checkbox"/> Sample Blank		<input type="checkbox"/> Auto rerun			

Define / Edite Chemestries

Chem	CHDL	N°		Sample Type	Serum
Chemistry	HDL	Print name			

Linearity range (standard)	* <input type="text"/>	* <input type="text"/>	Linearity Limit	* <input type="text"/>
Linearity range (Decreased)	* <input type="text"/>	* <input type="text"/>	Substrate Depletion	<input type="text"/>
Linearity range (Increased)	* <input type="text"/>	* <input type="text"/>	Mixed Blank Abs	* <input type="text"/> <input type="text"/>
R1 Blank Abs	* <input type="text"/>	* <input type="text"/>	On-borad Stability	* <input type="text"/> Jour
Blank Response	* <input type="text"/>	* <input type="text"/>	Reagent Alarm Limit	<input type="text"/>
Twin Chemistry	<input type="text"/>		<input type="checkbox"/> Enzyme Linear Extension	

Prozone Check Rtae Check

Q1	<input type="text"/>	Q2	<input type="text"/>	Q3	<input type="text"/>	Q4	<input type="text"/>	PC	<input type="text"/>	ABS	<input type="text"/>
----	----------------------	----	----------------------	----	----------------------	----	----------------------	----	----------------------	-----	----------------------

Calibration setup

Chem

Calibrator	Conc.	Pos	Lot No.
Water	0	W	
Cal1	xxx	xxx	

Calibration settings

Math Model	<input type="text" value="Two-point linear"/>		
Factor	<input type="text"/>	Replicates	<input type="text" value="2"/>

Acceptance Limits

Cal Time	<input type="text"/>	Heure	
Slope Diff	<input type="text"/>	SD	<input type="text"/>
Sensitivity	<input type="text"/>	Reproductb.	<input type="text"/>
Deter Coeff	<input type="text"/>		

Auto calib.

Bottle Changed
 Lot Changed
 Cal Time

*Values entered by the operator