


# BILIRUBIN DIRECT 4+1

## IMPROVED PRODUCT

Ref.: BITD-XXXX  
BIDI-XXXX

### APPLICATION PRESTIGE 24i - PROPOSAL

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use

- Instruction for use** :   
**Working temperature** : 37°C  
**Preparation** : Reagent 1: Ready for use  
 Reagent 2: Ready for use

Item Name		XXX	BIDI	
DATA INFORMATION				
UNITS	mg/dL			
DECIMALS	2			
ANALYSIS				
TYPE	END			
Main W Length 1	546			
Sub W Length 1				
METHOD				
CORR	SLOPE	Y=	INTER	
	1		X+ -0.05	
CALIBRATION				
TYPE	Linear			
#1	xxx	#4		
#2		#5		
#3		#6		
NORMAL RANGE				
	MALE		FEMALE	
	Low	High	Low	High
Serum	xxx	xxx	xxx	xxx
Urine				
Plasma				
CSF				
Dialysis				
other				

---

Item Name		XXX	BIDI
ASPIRATION			
KIND	<input type="radio"/> single <input checked="" type="radio"/> double		
SAMPLE	VOLUME	μL	
REAGENT 1 vol	25		
REAGENT 2 vol	50		
DATA PROCESS			
READ	Start	End	
Main	36	38	
Sub	29	31	
ABSORBANCE LIMIT			
	Low	3.000	
	High	3.000	
FACTOR			
Blank correction	0.8182		
Endpoint limit		2	
Linear check (%)		0	
Dilution			
Diluent	<input type="radio"/> 99: Dil 1 <input type="radio"/> 100: Dil 2		
PROZONE CHECK			
	start	End	Limit %
First			low high
Second			<input type="radio"/> <input type="radio"/>
third			<input type="radio"/> <input type="radio"/>

---

Item name		XXX	BIDI
Auto Rerun SW			
<input checked="" type="radio"/> ON <input type="radio"/> OFF		Auto rerun condition (Absorbance)	
		ABS. range	
		lower	<input type="radio"/> ON <input type="radio"/> OFF
		high	<input type="radio"/> ON <input type="radio"/> OFF
Auto Rerun Range (result)			
<input checked="" type="radio"/> ON <input type="radio"/> OFF		Prozone range	<input type="radio"/> ON <input type="radio"/> OFF
	lower	higher	
Serum	<0.08	<10.55	
Urine			
Plasma			
CSF			
Dialysis			
other			

XXX: enter data by the user

(05/2019)  
AA-PRESTIGE24i-BIDI 2  
(Improved Product)