


# CK NAC SL

## TWO REAGENT PROCEDURE

Réf. : CKSL-XXXX

**APPLICATION PRESTIGE 24i - PROPOSAL**

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use

**Instruction for use** : 

**Working temperature** : 37°C

Item Name : XXX CKSL		CALIBRATION																																									
DATA INFORMATION		TYPE : Linear																																									
UNITS : U/L	DECIMALS : 0	Standard																																									
ANALYSIS		<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td>#1</td><td>xxx</td><td>#4</td><td></td></tr> <tr> <td>#2</td><td></td><td>#5</td><td></td></tr> <tr> <td>#3</td><td></td><td>#6</td><td></td></tr> </table>		#1	xxx	#4		#2		#5		#3		#6																													
#1	xxx	#4																																									
#2		#5																																									
#3		#6																																									
TYPE : RATE		NORMAL RANGE																																									
Main W.Length 1 : 340	Sub W.Length 2	<table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th></th><th colspan="2">MALE</th><th colspan="2">FEMALE</th></tr> <tr> <th></th><th>Low</th><th>High</th><th>Low</th><th>High</th></tr> </thead> <tbody> <tr> <td>Serum</td><td>xxx</td><td>xxx</td><td>xxx</td><td>xxx</td></tr> <tr> <td>Urine</td><td></td><td></td><td></td><td></td></tr> <tr> <td>Plasma</td><td></td><td></td><td></td><td></td></tr> <tr> <td>CSF</td><td></td><td></td><td></td><td></td></tr> <tr> <td>Dialysis</td><td></td><td></td><td></td><td></td></tr> <tr> <td>other</td><td></td><td></td><td></td><td></td></tr> </tbody> </table>			MALE		FEMALE			Low	High	Low	High	Serum	xxx	xxx	xxx	xxx	Urine					Plasma					CSF					Dialysis					other				
	MALE		FEMALE																																								
	Low	High	Low	High																																							
Serum	xxx	xxx	xxx	xxx																																							
Urine																																											
Plasma																																											
CSF																																											
Dialysis																																											
other																																											
METHOD		CORR : SLOPE : INTER																																									
Y = 1	X + 0																																										

---

Item Name : XXX CKSL		ASPIRATION																
KIND : <input type="radio"/> single <input checked="" type="radio"/> double		DATA PROCESS																
Sample VOLUME : 12 $\mu$ L	REAGENT 1 vol : 240	READ	Absorbance limit															
REAGENT 2 vol : 60		<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <th></th><th>Start</th><th>End</th></tr> <tr> <td>Main</td><td>41</td><td>48</td></tr> <tr> <td>Sub</td><td></td><td></td></tr> </table>		Start	End	Main	41	48	Sub			<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td>Low</td><td>-3.000</td></tr> <tr> <td>High</td><td>3.000</td></tr> </table>	Low	-3.000	High	3.000		
	Start	End																
Main	41	48																
Sub																		
Low	-3.000																	
High	3.000																	
Third mix : <input type="radio"/> OFF <input type="radio"/> ON		FACTOR																
R1 Blank : <input type="radio"/> Water-blank <input type="radio"/> R1-blank-1	Dilution	Blank correction : 1																
Monitor		Endpoint limit : 0																
0 level point : 1	<table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th></th><th>start</th><th>End</th><th>Limit %</th></tr> </thead> <tbody> <tr> <td>First</td><td></td><td></td><td>low <input type="radio"/> high <input type="radio"/></td></tr> <tr> <td>Second</td><td></td><td></td><td><input type="radio"/></td></tr> <tr> <td>third</td><td></td><td></td><td><input type="radio"/></td></tr> </tbody> </table>		start	End	Limit %	First			low <input type="radio"/> high <input type="radio"/>	Second			<input type="radio"/>	third			<input type="radio"/>	Linear check (%) : 40
		start	End	Limit %														
First				low <input type="radio"/> high <input type="radio"/>														
Second			<input type="radio"/>															
third			<input type="radio"/>															
SPAN : 3	PROZONE CHECK																	

---

Item name : XXX CKSL		Auto rerun condition (Absorbance)																						
Auto Rerun SW : <input checked="" type="radio"/> ON <input type="radio"/> OFF		ABS. range																						
Auto Rerun Range (result) : <input checked="" type="radio"/> ON <input type="radio"/> OFF		<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td>lower</td><td><input type="radio"/> ON <input type="radio"/> OFF</td></tr> <tr> <td>high</td><td><input type="radio"/> ON <input type="radio"/> OFF</td></tr> </table>		lower	<input type="radio"/> ON <input type="radio"/> OFF	high	<input type="radio"/> ON <input type="radio"/> OFF																	
lower	<input type="radio"/> ON <input type="radio"/> OFF																							
high	<input type="radio"/> ON <input type="radio"/> OFF																							
<table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th></th><th>lower</th><th>higher</th></tr> </thead> <tbody> <tr> <td>Serum</td><td>10</td><td>1714</td></tr> <tr> <td>Urine</td><td></td><td></td></tr> <tr> <td>Plasma</td><td></td><td></td></tr> <tr> <td>CSF</td><td></td><td></td></tr> <tr> <td>Dialysis</td><td></td><td></td></tr> <tr> <td>other</td><td></td><td></td></tr> </tbody> </table>			lower	higher	Serum	10	1714	Urine			Plasma			CSF			Dialysis			other			Prozone range : <input type="radio"/> ON <input checked="" type="radio"/> OFF	
	lower	higher																						
Serum	10	1714																						
Urine																								
Plasma																								
CSF																								
Dialysis																								
other																								

XXX: enter data by the user