

# IRON FERENE

## Two REAGENT PROCEDURE

Ref : FEFE-XXXX

**APPLICATION PRESTIGE 24i - PROPOSAL**

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use

Instruction for use : 

Working temperature : 37°C

Item Name : XXX FEFE																																								
<b>DATA INFORMATION</b> UNITS : µg/dL DECIMALS : 0	<b>CALIBRATION</b> TYPE : Linear Standard #1 : xxx #4 : #2 : #5 : #3 : #6 :																																							
<b>ANALYSIS</b> TYPE : END Main W.Length 1 : 570 Sub W.Length 2 : 700 METHOD :	<b>NORMAL RANGE</b> <table border="1" style="width: 100%; text-align: center;"> <thead> <tr> <th rowspan="2"></th> <th colspan="2">MALE</th> <th colspan="2">FEMALE</th> </tr> <tr> <th>Low</th> <th>High</th> <th>Low</th> <th>High</th> </tr> </thead> <tbody> <tr> <td>Serum</td> <td>xxx</td> <td>xxx</td> <td>xxx</td> <td>xxx</td> </tr> <tr> <td>Urine</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Plasma</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>CSF</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Dialysis</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>other</td> <td></td> <td></td> <td></td> <td></td> </tr> </tbody> </table>		MALE		FEMALE		Low	High	Low	High	Serum	xxx	xxx	xxx	xxx	Urine					Plasma					CSF					Dialysis					other				
	MALE		FEMALE																																					
	Low	High	Low	High																																				
Serum	xxx	xxx	xxx	xxx																																				
Urine																																								
Plasma																																								
CSF																																								
Dialysis																																								
other																																								
CORR : SLOPE : INTER : Y = 1 X+ 0																																								


  

Item Name : XXX FEFE																									
<b>ASPIRATION</b> KIND : <input type="radio"/> single <input type="radio"/> double Sample : 30 REAGENT 1 vol : 240 µL REAGENT 2 vol : 60	<b>DATA PROCESS</b> READ : Start : End : Main : 52 54 Sub : 29 31 Absorbance limit : Low : -3.000 High : 3.000 FACTOR : Blank correction : 0.8182 Endpoint limit : 2 Linear check (%) : 0																								
Third mix : <input type="radio"/> OFF <input type="radio"/> ON R1 Blank : <input type="radio"/> Water blank <input type="radio"/> R1 blank-1	<b>Dilution</b> Diluent : <input type="radio"/> 99:Di 1 <input type="radio"/> 100:Di 2																								
<b>Monitor</b> 0 level point : 1 SPAN : 3	<b>PROZONE CHECK</b> <table border="1" style="width: 100%; text-align: center;"> <thead> <tr> <th></th> <th>start</th> <th>End</th> <th>Limit %</th> <th>low</th> <th>high</th> </tr> </thead> <tbody> <tr> <td>First</td> <td></td> <td></td> <td></td> <td><input type="radio"/></td> <td><input type="radio"/></td> </tr> <tr> <td>Second</td> <td></td> <td></td> <td></td> <td><input type="radio"/></td> <td><input type="radio"/></td> </tr> <tr> <td>third</td> <td></td> <td></td> <td></td> <td><input type="radio"/></td> <td><input type="radio"/></td> </tr> </tbody> </table>		start	End	Limit %	low	high	First				<input type="radio"/>	<input type="radio"/>	Second				<input type="radio"/>	<input type="radio"/>	third				<input type="radio"/>	<input type="radio"/>
	start	End	Limit %	low	high																				
First				<input type="radio"/>	<input type="radio"/>																				
Second				<input type="radio"/>	<input type="radio"/>																				
third				<input type="radio"/>	<input type="radio"/>																				

Item name : XXX FEFE																						
Auto Rerun SW : <input type="radio"/> ON <input type="radio"/> OFF Auto Rerun Range (result) : <input type="radio"/> ON <input type="radio"/> OFF	Auto rerun condition (Absorbance) ABS. range : lower <input type="radio"/> ON <input type="radio"/> OFF high <input type="radio"/> ON <input type="radio"/> OFF Prozone range : <input type="radio"/> ON <input type="radio"/> OFF																					
<table border="1" style="width: 100%; text-align: center;"> <thead> <tr> <th></th> <th>lower</th> <th>higher</th> </tr> </thead> <tbody> <tr> <td>Serum</td> <td>20</td> <td>1000</td> </tr> <tr> <td>Urine</td> <td></td> <td></td> </tr> <tr> <td>Plasma</td> <td></td> <td></td> </tr> <tr> <td>CSF</td> <td></td> <td></td> </tr> <tr> <td>Dialysis</td> <td></td> <td></td> </tr> <tr> <td>other</td> <td></td> <td></td> </tr> </tbody> </table>		lower	higher	Serum	20	1000	Urine			Plasma			CSF			Dialysis			other			
	lower	higher																				
Serum	20	1000																				
Urine																						
Plasma																						
CSF																						
Dialysis																						
other																						

XXX: enter data by the user

 Modification from the previous version