


LIPASE SL

Ref : LPSL-XXXX

APPLICATION PRESTIGE 24i - PROPOSAL

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use

Instruction for use : 

Working temperature : 37°C

Item Name		XXX	LIPASE	
DATA INFORMATION				
UNITS	U/L			
DECIMALS	0			
ANALYSIS				
TYPE	RATE			
Main W.Length 1	570			
Sub W.Length 2				
METHOD				
CORR	SLOPE	INTER		
Y=	1.000	X+	0	
CALIBRATION				
TYPE	Linear			
Standard				
#1	xxx	#4		
#2		#5		
#3		#6		
NORMAL RANGE				
	MALE		FEMALE	
	Low	High	Low	High
Serum	XXX	XXX	XXX	XXX
Urine				
Plasma				
CSF				
Dialysis				
other				
Item Name XXX LIPASE				
ASPIRATION				
KIND	<input type="radio"/> single <input checked="" type="radio"/> double			
VOLUME				
Sample	4			
REAGENT 1 vol	200	µL		
REAGENT 2 vol	120			
DATA PROCESS				
READ				
	Start	End		
Main	39	50		
Sub				
Absorbance limit				
Low	-3.000			
High	3.000			
FACTOR				
Blank correction	1			
Endpoint limit		0		
Linear check (%)		40		
Third mix	<input type="radio"/> OFF <input type="radio"/> ON			
R1 Blank	<input type="radio"/> Water-blank <input type="radio"/> R1-blank-1			
Dilution				
Diluent	<input type="radio"/> 99:DI 1 <input type="radio"/> 100:DI 2			
PROZONE CHECK				
	start	End	Limit %	
First				
Second				
third				
	low	high		
	<input type="radio"/>	<input type="radio"/>		
	<input type="radio"/>	<input type="radio"/>		
Item name XXX LIPASE				
Auto rerun condition (Absorbance)				
Auto Rerun SW	<input type="radio"/> ON <input type="radio"/> OFF			
Auto Rerun Range (result)	<input type="radio"/> ON <input type="radio"/> OFF			
ABS. range				
lower	<input type="radio"/> ON <input type="radio"/> OFF			
high	<input type="radio"/> ON <input type="radio"/> OFF			
Prozone range		<input type="radio"/> ON <input type="radio"/> OFF		
lower	higher			
Serum	xxx	xxx		
Urine				
Plasma				
CSF				
Dialysis				
other				

XXX: enter data by the user