

APPLICATION - RESPONS 920 - ERBA XL200 / XL-180 - EM200 - PROPOSAL

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use.

Consult Instruction for Use

TEST DETAILS

Test	: AMY	Auto Rerun	<input type="checkbox"/>
Report Name	: AMYLASE SL	Online calibration	<input type="checkbox"/>
Unit	: U/L	Cuvette Wash	<input type="checkbox"/>
Wavelength-Primary	: 405	Total reagents	: 1
Assay Type	: RATE-A	Reagent 1	: AMSL R1
M1 Start	: 0	Reagent 2	:
M2 Start	: 5	M1 End	: 0
Sample Replicates	: 1	M2 End	: 14
Control Replicates	: 1	Standard replicates	: 3
Reaction Direction	: Increasing	Control Interval	: 0
Prozone Limit %	: 0	React. Abs. Limit	: *
Linearity Limit %	: 0	Prozone Check	: lower
Technical Minimum	: *	Delta Abs./Min.	: 0.0000
Y= aX + b	a= : 1.0000	Technical Maximum	: *
		B=	: 0.0000

TEST VOLUMES

Test	: AMY
Sample Type	: Serum
Sample Volumes	
Normal	: 2.00 µL
Increase	: * µL
Decrease	: * µL
Standard Volume	: 2.00 µL
reagent Volumes and Stirrer Speed	
RGT-1 Volume	: 200 µL
RGT-2 Volume	: 0 µL
R1 Stirrer Speed	: medium
R2 Stirrer Speed	: 0

Sample Types	
<input checked="" type="checkbox"/>	Serum
<input type="checkbox"/>	Urine
<input type="checkbox"/>	CSF
<input checked="" type="checkbox"/>	Plasma
<input type="checkbox"/>	Whole blood
<input type="checkbox"/>	Other

REFERENCE RANGES

Test	: AMY	
Sample Type	: Serum	
Reference Range	: DEFAULT	
Category	: *	
Reference Range		
	Lower Limit	Lower Limit
	U/L	U/L
Normal	: *	: *
Panic	: *	: *

Sample Types	
<input checked="" type="checkbox"/>	Serum
<input type="checkbox"/>	Urine
<input type="checkbox"/>	CSF
<input checked="" type="checkbox"/>	Plasma
<input type="checkbox"/>	Whole blood
<input type="checkbox"/>	Other

*: Values entered by the operator