

**APPLICATION - RESPONS 920 - ERBA XL200 / XL-180 - EM200 - PROPOSAL**

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use.

Consult Instruction for Use

**TEST DETAILS**

Test	:	LIP	Auto Rerun	<input type="checkbox"/>
Report Name	:	LIPASE SL	Online calibration	<input type="checkbox"/>
Unit	:	U/L	Cuvette Wash	<input type="checkbox"/>
Wavelength-Primary	:	578	Total reagents	: 2
Assay Type	:	RATE-A	Curve Type	: Linear
M1 Start	:	0	M1 End	: 0
M2 Start	:	22	M2 End	: 31
Sample Replicates	:	1	Standard replicates	: 3
Control Replicates	:	1	Control Interval	: 0
Reaction Direction	:	Increasing	React. Abs. Limit	: *
Prozone Limit %	:	0	Prozone Check	: Lower
Linearity Limit %	:	0	Delta Abs./Min.	: 0.0000
Technical Minimum	:	*	Technical Maximum	: *
Y= aX + b	a=	: 1.0000	B=	: -2.0000

**TEST VOLUMES**

Test	:	LIP	Dilution Ratio	:	1 X
Sample Type	:	Serum	Dilution Ratio	:	* X
<b>Sample Volumes</b>					
Normal	:	3.00 µL	Dilution Ratio	:	* X
Increase	:	* µL	Dilution Ratio	:	* X
Decrease	:	* µL			
Standard Volume	:	3.00 µL			
<b>reagent Volumes and Stirrer Speed</b>					
RGT-1 Volume	:	150 µL	R1 Stirrer Speed	:	medium
RGT-2 Volume	:	90 µL	R2 Stirrer Speed	:	medium

Sample Types	
<input checked="" type="checkbox"/>	Serum
<input type="checkbox"/>	Urine
<input type="checkbox"/>	CSF
<input checked="" type="checkbox"/>	Plasma
<input type="checkbox"/>	Whole blood
<input type="checkbox"/>	Other

**REFERENCE RANGES**

Test	:	LIP
Sample Type	:	Serum
Reference Range	:	DEFAULT
Category	:	*
<b>Reference Range</b>		
	Lower Limit	Lower Limit
	U/L	U/L
Normal	:	*
Panic	:	*

Sample Types	
<input checked="" type="checkbox"/>	Serum
<input type="checkbox"/>	Urine
<input type="checkbox"/>	CSF
<input checked="" type="checkbox"/>	Plasma
<input type="checkbox"/>	Whole blood
<input type="checkbox"/>	Other

\*: Values entered by the operator