

APPLICATION - RESPONS 920 - ERBA XL200 / XL-180 - EM200 - PROPOSAL

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use.

Consult Instruction for Use

TEST DETAILS

Test	:	PROT U	Auto Rerun	<input type="checkbox"/>
Report Name	:	MICROPROTEIN PLUS	Online calibration	<input type="checkbox"/>
Unit	:	mg/dL	Decimal Places	: 1
Wavelength-Primary	:	600	Secondary	: -
Assay Type	:	1-Point	Curve Type	: Linear
M1 Start	:	0	M1 End	: 0
M2 Start	:	17	M2 End	: 17
Sample Replicates	:	1	Standard replicates	: 3
Control Replicates	:	1	Control Interval	: 0
Reaction Direction	:	Increasing	React. Abs. Limit	: *
Prozone Limit %	:	0	Prozone Check	: lower
Linearity Limit %	:	0	Delta Abs./Min.	: 0.0000
Technical Minimum	:	*	Technical Maximum	: *
Y= aX + b	a=	1.0000	B=	0.0000
			Reagent 1	: PRTU R1
			Reagent 2	:
			Cuvette Wash	<input type="checkbox"/>
			Total reagents	: 1

TEST VOLUMES

Test	:	PROT U
Sample Type	:	Urine
Sample Volumes		
Normal	:	4.00 µL
Increase	:	* µL
Decrease	:	* µL
Standard Volume	:	4.00 µL
reagent Volumes and Stirrer Speed		
RGT-1 Volume	:	200 µL
RGT-2 Volume	:	0 µL
R1 Stirrer Speed	:	high
R2 Stirrer Speed	:	0

Sample Types	
<input type="checkbox"/>	Serum
<input checked="" type="checkbox"/>	Urine
<input type="checkbox"/>	CSF
<input type="checkbox"/>	Plasma
<input type="checkbox"/>	Whole blood
<input type="checkbox"/>	Other

REFERENCE RANGES

Test	:	PROT U
Sample Type	:	Urine
Reference Range	:	DEFAULT
Category	:	*
Reference Range		
	Lower Limit	Lower Limit
	mg/dL	mg/dL
Normal	:	*
Panic	:	*

Sample Types	
<input type="checkbox"/>	Serum
<input checked="" type="checkbox"/>	Urine
<input type="checkbox"/>	CSF
<input type="checkbox"/>	Plasma
<input type="checkbox"/>	Whole blood
<input type="checkbox"/>	Other

*: Values entered by the operator