

APPLICATION - RESPONS 920 - ERBA XL200 / XL-180 - EM200 - PROPOSAL

This application is intended to serve as a guide for using the referenced ELITechGroup Reagent on this instrument system only. It is recommended that the user validate this application prior to routine use.

Consult Instruction for Use

TEST DETAILS

Test	:	TRIG		Auto Rerun	<input type="checkbox"/>	
Report Name	:	TRIGLYCERIDES SL MONO		Online calibration	<input type="checkbox"/>	
Unit	:	mg/dL	Decimal Places	:	0	
Wavelength-Primary	:	505	Secondary	:	-	
Assay Type	:	1-Point	Curve Type	:	Linear	
M1 Start	:	0	M1 End	:	0	
M2 Start	:	35*	M2 End	:	35*	
Sample Replicates	:	1	Standard replicates	:	3	
Control Replicates	:	1	Control Interval	:	0	
Reaction Direction	:	Increasing	React. Abs. Limit	:	*	
Prozone Limit %	:	0	Prozone Check	:	lower	
Linearity Limit %	:	0	Delta Abs./Min.	:	0.0000	
Technical Minimum	:	*	Technical Maximum	:	*	
Y= aX + b	a=	1.0000	B=	:	0.0000	
				Reagent 1	:	TGML R1
				Reagent 2	:	

TEST VOLUMES

Test : TRIG
Sample Type : Serum

Sample Volumes

Normal	:	2.00 µL	Dilution Ratio	:	1 X
Increase	:	* µL	Dilution Ratio	:	* X
Decrease	:	* µL	Dilution Ratio	:	* X
Standard Volume	:	2.00 µL			

reagent Volumes and Stirrer Speed

RGT-1 Volume	:	200 µL	R1 Stirrer Speed	:	high
RGT-2 Volume	:	0 µL	R2 Stirrer Speed	:	0

Sample Types

- Serum
- Urine
- CSF
- Plasma
- Whole blood
- Other

REFERENCE RANGES

Test : TRIG
Sample Type : Serum
Reference Range : DEFAULT
Category : *

Reference Range

		Lower Limit		Lower Limit
		mg/dL		mg/dL
Normal	:	*	:	*
Panic	:	*	:	*

Sample Types

- Serum
- Urine
- CSF
- Plasma
- Whole blood
- Other

* XL-180 : 34

*: Values entered by the operator